

QUICK GUIDE

- TREATMENT APPLICATIONS



SET UP

Apply 45 inch belly strap to the horse as shown. All applications will originate from this initial set up. Place Velcro backed device pouch to the belly strap. Then place the Equicurrent device in the pouch and insert one or two lead wires into the top of the device as the treatment requires. (ie. If treating both legs you will need, one wire to each leg.)



LEG APPLICATION


After completing the set up process, plug 18 inch extensions to the lead wires. (Use 18 inch extensions for front legs and 36 inch extensions to reach rear legs.) Slip the dual sleeve over the hoof as shown. This application can be used for ankles, knees, pasterns, and feet. Always wet the area to be treated with water or conductive spray, as well as the sleeve. Plug the extensions into the pigtails attached to the sleeve and wrap with the 22 inch versatile wraps to keep good contact as shown. Number of treatments can vary based on age and severity of injury.



SUSPENSORY/TENDON

After completing the set up process, plug 18 inch extensions to the lead wires. (Use 18 inch extensions for front legs and 36 inch extensions to reach rear legs.) Use the 2x6 patches, as shown. Always wet the area to be treated with water or conductive spray, as well as the patches. Plug the extensions into the pigtails attached to the patches and wrap with the 22 inch versatile wraps to keep good contact as shown. Tendons can be treated with some flexibility.


example 1: You may treat using the dual sleeve on the injured tendon.
example 2: You may wrap the 4x7 patches around the tendon, placing one above the other. Number of treatments can vary based on age and severity of injury.

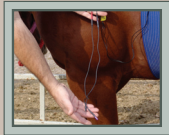

HIND QUARTER SORENESS


After completing the set up process, use the 22 inch versatile wrap above the hock to hold the 36 inch extension in place. Plug in the 36 inch extensions into lead wires, slip single sleeve over the hoof covering the hair line and plug in the red lead into the sleeve. Always wet the area to be treated with water or conductive spray, as well as the sleeve and patches. Wrap using the 22 inch wrap as previously shown. With the black lead, plug in 4x7 patches to apply to areas of soreness. You may use one or two patches as shown. If using 2 patches you will need the splitter wires. Simply attach the splitter wires to the male end of the black lead wire. Always wet the area to be treated with water or conductive spray, as well as the patches. At this point, you will need to tape the patches on. (duct tape, Elastikon, etc... anything that will accomplish this task)

**note:* When treating the hind quarters, be sure the treat both sides of the horse. You do this by just duplicating the process on the other side of the horse. Number of treatments can vary based on age and severity of injury.

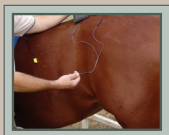

FRONT & REARTING POINTS

After completing the set up process. Install single sleeve, over the hoof, being sure to cover beyond the hairline as shown. Do this to both feet, front or rear depending on the application using. Install 36 inch extension to lead wire if working rear ting points and 18 inch extensions if treating front ting points. Plug extensions into pigtail. Always wet the area to be treated with water or conductive spray, as well as the sleeve and wrap with the 22 inch versatile wraps to keep good contact as shown. This application is done using only 1 lead wire from the device (ie. 1 pin to the right hoof and 1 pin to the left) this will stimulate and balance the meridians. Number of treatments can vary based on age and severity of injury.

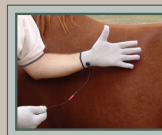

GLOVES

Gloves are used for many applications. Especially hard to reach or hard to treat areas. You can be very creative with the gloves. You can treat wherever you can touch. If you have purchased the deluxe kit, start by clipping the device to your body, then plug in one set of lead wires. ***Important:** You will need to use latex gloves under the Equi-gloves. This is so the current will be transferred to the horse and not you. Next slip on the gloves over the latex, then plug in the lead wires to the pigtails on the gloves. Next wet the gloves with water or conductive spray, as well as the area on the horse you want to treat. When treating, press firmly with your palms and fingers to stay in contact with the area you are treating. Re-wet gloves and horse as needed. Number of treatments can vary based on age and severity of injury.



HOCKS

After completing the set up process. Slip dual sleeves over the hoof up to the hock and wet the hock as shown. Next pull the sleeve in place over the entire hock joint and wet the sleeve. Install 36 inch extension to the lead wire from the device and wrap as shown to keep contact with the area. Number of treatments can vary based on age and severity of injury.

SETTINGS & TREATMENT DURATIONS

- » **Settings** – Follow the frequency and current setting guidelines as outlined in the manual.
- » **Treatment Durations** – It is recommended to treat a minimum or 20 minutes per application. But, we have found that treating 30 minutes to a hour is more beneficial depending on time constraints.

For any additional treatment application, please contact your regional sales rep at www.equicurrent.com under contact us.